Instructions for Use QuickTrach

Emergency Cricothyrotomy Device

Pre-assembly
Open the package, remove the device, and familiarize yourself with its contents.

Step One

Place the patient in a supine position. Assure stable positioning of the neck region (place a pillow or piece of clothing under the patient's shoulders) and hyperextend the neck.

Score the larynx laterally between the thumb and forefinger. Find the cricothyroid ligament (in the midline between the thyroid cartilage and the cricoid cartilage). This is the puncture site.

Step Two

Firmly hold the device and puncture the cricothyroid ligament at a 90° angle. (Note: Because of the sharp tip and conical shape of the needle, an incision of the skin with a scalpel is not necessary. The opening of the trachea is achieved by dilating through the skin. This reduces the risk of bleeding as only the smallest necessary opening is made.

Step Three

After puncturing the cricothyroid ligament, check the entry of the needle into the trachea by aspirating air through the syringe. If air is present, the needle is within the trachea*. Now, change the angle of insertion to 60° and advance the device forward into the trachea to the level of the stopper. The stopper reduces the risk of inserting the needle too deeply and causing damage to the rear wall of the trachea.

Step Four

Remove the stopper. After the stopper is removed, be careful not to advance the device further with the needle still attached.

Step Five

Hold the needle and syringe firmly and slide only the plastic cannula along the needle into the trachea until the flange rests on the neck. Carefully remove the needle and syringe. Next, secure the cannula with the neck tape, apply the connecting tube to the 15mm connection, and connect the other end to the resuscitation bag or ventilation circuit.

*Warning

Should no aspiration of air be possible in Step Three because of an extremely thick neck, it is possible to remove the stopper and carefully insert the needle further until entrance into the trachea is made. Once this is verified, continue as in Step Five.

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